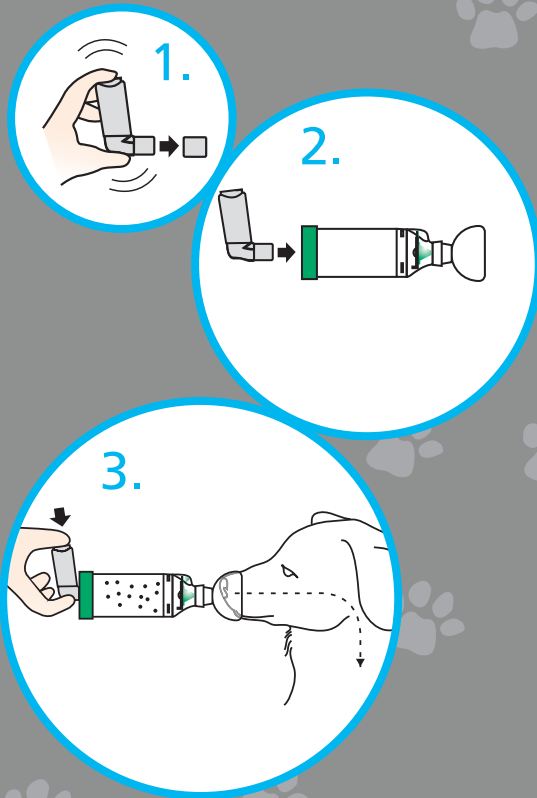
Instructions for Use

1. Remove cap and shake inhaler
2. Insert inhaler into back of chamber and gently apply the mask to your dog's face
3. Press the inhaler to release the medication when the **Flow-Vu*** Inhalation Indicator is moving toward your dog and hold the mask in place for 5–6 breaths

Tips and tricks

Here are some tips and tricks from some other **AeroDawg*** Chamber users.

- ▶ Reward your dog with treats before and/or after treatment
- ▶ Swaddle your dog in a towel or blanket
- ▶ Slowly introduce the **AeroDawg*** Chamber. Start with the mask only, getting your dog used to the feel of it
- ▶ Use the **Flow-Vu*** Inhalation Indicator to help ensure your dog has inhaled their medication
- ▶ Reference the numerous videos on YouTube
- ▶ Be patient — don't give up!

725 Third Street, London,
Ontario, Canada, N5V 5G4
+1-519-455-7060 F: +1-519-455-7858
1-800-465-3296

www.aerodawg.com

Does your dog suffer from respiratory symptoms?

101607-001 Rev. C * Trade-mark and registered trade-name of Trudell Medical International. © TMI 2015. All rights reserved.

Trudell Medical International

Chronic bronchitis is one of the most common respiratory diseases in dogs

Chronic bronchitis in dogs is caused by inflammation of the airways, resulting in chronic cough and exercise intolerance. Common signs of chronic bronchitis include daily coughing, gagging or swallowing after the cough, difficulty breathing or wheezing for a duration of 2-3 months or longer.

Treatment

The **AeroDawg*** Canine Aerosol Chamber is designed to deliver Metered Dose Inhaler medication as recommended by a veterinarian to dogs with chronic bronchitis, laryngeal paralysis or tracheal collapse.

When the **AeroDawg*** Chamber is used with the inhaler it allows your dog to breathe normally and inhale the medication into their lungs, where it is needed.

Can be used with all types of Metered Dose Inhalers

"I got the **AeroDawg*** as a last effort before putting my five year old dog down. I didn't know if it would work, but I was willing to try it just to keep her a little longer. She can breathe again, and I just wanted to thank you for making this product. If we don't remember to give her the **AeroDawg*** she will go sit under the shelf where we keep it until she gets it. Thanks from my dog Penelope and me!"

Hannah, Dog Owner

AeroDawg* 

Canine Aerosol Chamber

Flow-Vu* Inhalation Indicator

Takes the guesswork out of administering your dog's inhaler medication. The **Flow-Vu*** Inhalation Indicator moves toward your dog as they inhale and will only move if there is a good seal.

Inhalation Valve

Extra sensitive, low resistance valve is designed to respond to shallow, low volume breaths

Anti-Static Chamber

Holds the medication allowing your dog time to inhale

2 Mask Sizes

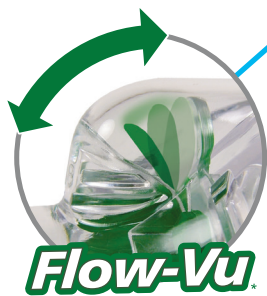
Each device is shipped with 2 different sized facemasks. The masks are specifically designed to reduce irritation to your dog's fur.

The **AeroDawg*** Canine Aerosol Chamber is designed to be used with a Metered Dose Inhaler (puffer) to deliver aerosol medication to canines with chronic bronchitis, chronic rhinitis, laryngeal paralysis or tracheal collapse.

The use of inhaled medication provides targeted delivery of medication to the lungs and reduces the side effects seen in dogs treated with systemic medications.¹ Therapeutic effect can often be achieved with only a fraction of the dose required for systemic delivery of the same drug.²

Flow-Vu* Inhalation Indicator

The **Flow-Vu*** Indicator moves with inhalation and provides the caregiver with visual confirmation of the dog inhaling their medication. This is important because some dogs hold their breath when first introduced to the mask. The **Flow-Vu*** Indicator also allows the caregiver to count breaths and ensure a proper facemask seal.

2 Mask Sizes

Each device is shipped with 2 different sized facemasks. The mask should cover the nose and mouth. However, if not possible apply the mask to the dogs nostrils while gently holding the muzzle closed.

Inhalation Valve

Extra sensitive, low resistance valve is designed to respond to shallow, low volume breaths

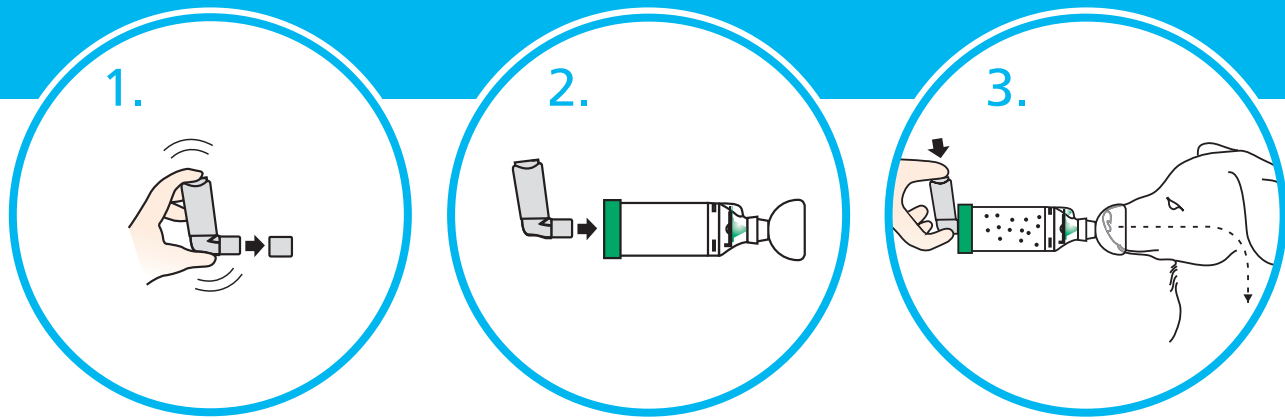
Anti-static Chamber

Holds the medication allowing the dog more time to inhale

Universal Backpiece

Can be used with all commonly prescribed Metered Dose Inhalers (Note: not designed for use with Dry Powder Inhalers)

Easy to Use

1. Remove cap and shake inhaler

2. Insert inhaler into back of chamber and gently apply the mask to the dog's face

3. Press the inhaler to release the medication when the **Flow-Vu*** Inhalation Indicator is moving toward the dog and hold the mask in place for 5–6 breaths

Metered Dose Inhaler Treatment

Adapted from: Padrid P. Use of inhaled medications to treat respiratory diseases in dogs and cats. Journal of the American Animal Hospital Association 42:165-169 (2006).

DRUG	DOSE	USAGE	INDICATION
Salbutamol • Available through different manufacturers	• 1-2 puffs (100 µg/puff) administered via AeroDawg* Chamber • Each Metered Dose Inhaler contains 200 puffs	• Use as needed, but not routinely • In emergency cases, salbutamol can be used q30 minutes for up to 4 to 6 hours without serious side effects	Immediate relief of bronchoconstriction
Fluticasone Propionate • Brand name: Flixotide [†] Evohaler [†] Metered Dose Inhaler • Available in three strengths: 50 µg, 125 µg and 250 µg/actuation	• Dogs <20 kg: 1 puff (125 µg) q12h administered via AeroDawg* Chamber • Dogs >20 kg: 1 puff (250 µg) q12h administered via AeroDawg* Chamber • Each Metered Dose Inhaler contains 120 puffs	Overlap with oral prednisolone for 2-3 weeks when starting since Flixotide [†] Evohaler [†] takes 1-2 weeks to reach peak effect	Long-term control of airway inflammation

NOTES:

- If the dog gets frightened when pressing the inhaler, press the inhaler before placing the mask on the dogs muzzle. Note: this means the dog will receive a lower dose and may then require a second administration to guarantee they receive an adequate amount of medication.⁴
- If the dog is treated with both fluticasone and salbutamol, the salbutamol should be given first.⁴
- Although a number of additional corticosteroid drugs have been produced for inhaled administration, fluticasone is preferred, because it is the most potent and has the longest half-life¹
- Some of the newer Metered Dose Inhalers include dose indicators. If not, owners are encouraged to keep track of puffs.

¹ Padrid P. Use of inhaled medications to treat respiratory diseases in dogs and cats. Journal of the American Animal Hospital Association 42:165-169 (2006).

² Labiris NR, Dolovich MB. Pulmonary drug delivery. Part II: the role of inhalant delivery devices and drug formulations in therapeutic effectiveness of aerosolized medications. Br J Clin Pharmacol 2003;56(6):600-612.

³ Olah GA. Which drugs are used to treat feline asthma? Plumbs Therapeutics Brief May 2015: 18-30.

⁴ Pernas GS. Feline asthma. Veterinary Focus. Vol 20 no 2, 2010: 10-17.

MD-634A-1215 * trade-marks and registered trade-marks of Trudell Medical International. † trade-marks and registered trade-marks of the respective companies. © TMI 2015. All rights reserved.

Trudell Medical International[†]

725 Third Street, London,
Ontario, Canada, N5V 5G4

Distributed by: **Breatheazy**[®]
respiratory relief for cats, dogs and horses

154 Worcester Road, Malvern, Worcestershire WR14 1AA,
United Kingdom Tel: +44 845 680 8975 Fax: +44 845 680 8976
enquiries@breatheazy.co.uk

AeroDawg 

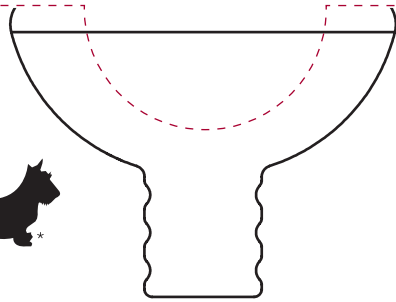
Canine Aerosol Chamber

www.aerodawg.co.uk


HOW TO USE: Print this template on letter paper, landscape. DO NOT SCALE. Print at 100% or disable page scaling. Cut along the dashed lines and remove inner shape. Insert your dog's nose and mouth into the front and side cut-outs to see which mask will fit best.

AeroDawg 


Canine Aerosol Chamber




SMALL

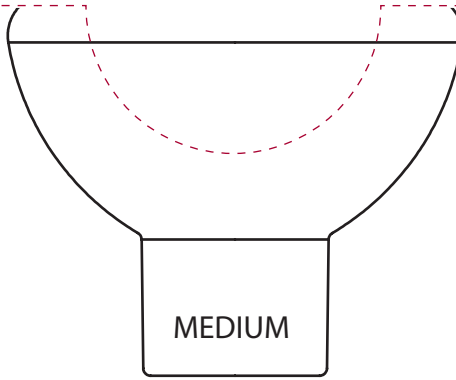MEDIUM

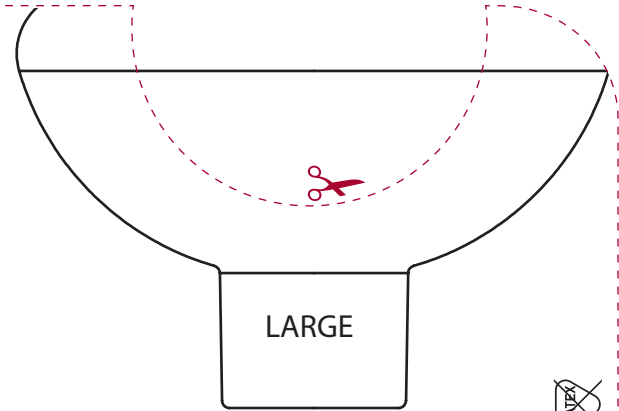LARGE

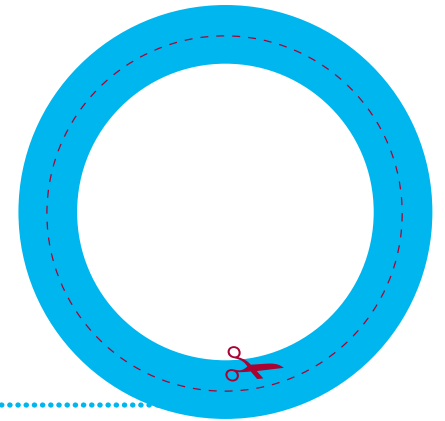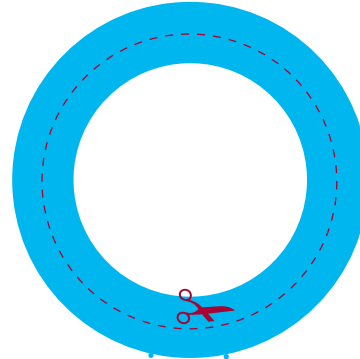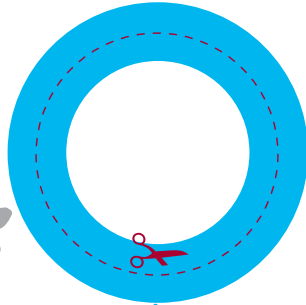Mask Sizer

Each device is shipped with 2 different sized masks.

AeroDawg* Chamber SMALL,
includes small and medium masks

AeroDawg* Chamber LARGE,
includes medium and large masks

How to Use

The mask selected should cover the nose and mouth. However, if not possible apply mask to the dog's nostrils while gently holding the muzzle closed.

Trudell Medical International[®]

www.aerodawg.com

MD-615A-1015 * trade-marks and registered trade-marks of Trudell Medical International. © TMI 2015. All rights reserved.

AeroKat

Feline Aerosol Chamber

www.aerokat.com

EN INTENDED USE

The chamber is intended to be used along with a metered dose inhaler to deliver aerosol medication to your cat or dog's lungs as prescribed by your veterinarian.

BEFORE USE

The chamber can be used directly out-of-package. Ensure these instructions and the instructions supplied with the inhaler have been read. Carefully examine the chamber and replace immediately if any defect is noticed. The chamber is shipped with 2 different sized masks. Select the mask that fits best and insert on frontpiece of chamber. The mask should cover the animal's nose and mouth but not eyes.

USAGE INSTRUCTIONS

1 Remove cap and shake the inhaler immediately before use as per the instructions supplied with it. **2** Insert the inhaler into the backpiece of the chamber and gently apply the mask to your animal's nostrils while gently holding the muzzle closed. **3** Press the inhaler to release the medication. Hold the mask in place for 5-6 breaths as the cat or dog inhales the medicine.

TIPS AND TRICKS:

• Slowly introduce the chamber. Start with the mask only, getting the animal used to the feel of it
• Reward the animal with treats before and / or after treatment
• Swaddle them in a towel or blanket
• If your cat or dog does not respond well to the sound of the inhaler, hold the chamber behind your back and press the inhaler and then immediately place mask on their face.

Notes:

- Use the ***Flow-Vu***[®] Inhalation Indicator to ensure a good seal, coordinate pressing the inhaler with inhalation and to count the number of breaths taken. The ***Flow-Vu***[®] Indicator moves toward your cat or dog as they inhale and only moves if there is a proper seal.
- Administer one (1) puff at a time. Follow instructions supplied by your veterinarian on the number of puffs to take and how long to wait before repeating.
- Wipe the animal's face with a damp cloth after treatment to remove medication residue.

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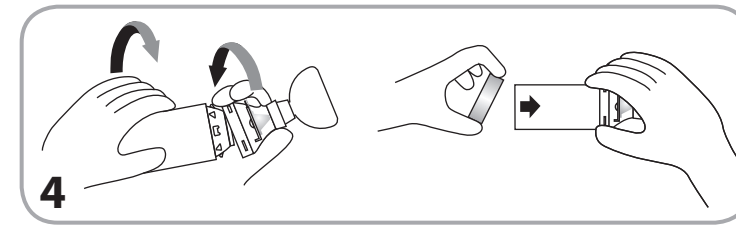
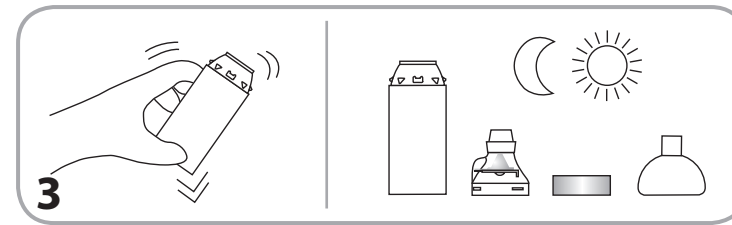
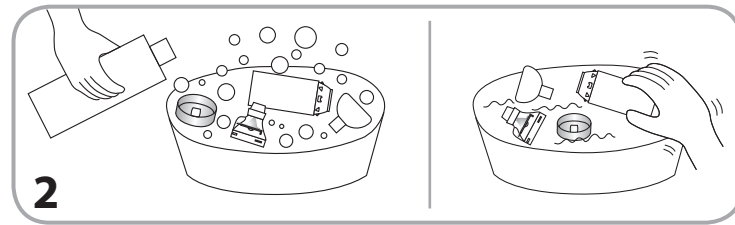
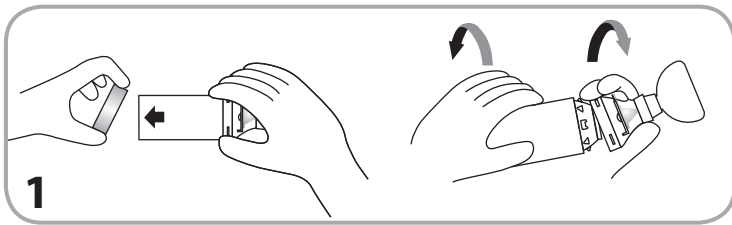
EN CLEANING INSTRUCTIONS

The chamber should be cleaned weekly.

1 Remove the backpiece (A). To detach the frontpiece (B), twist chamber as shown and then remove mask (C) from frontpiece. **2** Soak the parts for 15 minutes in a mild solution of liquid dish detergent and lukewarm clean water. Agitate gently and then rinse parts in clean water. **3** Shake out excess water and allow to air dry in a vertical position. Ensure parts are dry before reassembly. **4** To reassemble, fit the mask on the frontpiece and then place frontpiece on the end of the chamber and twist firmly until securely locked into position. Center the Alignment Feature (D) on the backpiece with the **Flow-Vu*** Indicator (E) as shown. Press firmly to attach the backpiece.

Notes

- If you notice medication build-up in your chamber, wash the inside of the chamber gently with a soft cloth.
- Do not boil, sterilize or clean in a dishwasher.
- Product should be replaced after 12 months of continuous use

Cautions

- Do not leave the chamber unattended with children.

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